

## Quidel Triage TOX delivers.

When choosing a rapid drug screen, there are many factors to consider: accuracy, ease-of-use, system interface capabilities and test selectivity.



**The instrumentation of the Quidel Triage TOX Drug Screen platform provides these features.**

### Reliable

- Built-in QC satisfies daily QC. External POS/NEG QC run once per month/new lot/shipment.
- Accurate/Calibrated: Patient sample is automatically analyzed against the lot-specific, multi-point calibration curve provided via the Reagent CODE CHIP™ Module.

### Flexible

- Customizable panels with Test Select.™

### Connected

- Qualitative POS/NEG results are displayed, printed and can be transmitted to the LIS.

**Automated. Calibrated. Connected.**

## Simplify your drug screening process.

1

Add urine sample to the Quidel Triage TOX DS Test using transfer pipette included in each kit.



2

Insert test device into Quidel Triage Meter.



3

Read results on screen, or press "Print" for a hard copy. Complete in approximately 15 minutes.



Please refer to Package Insert for complete instructions, limitations and warnings.

## Quidel Triage TOX Drug Screen Test



Rapid results for the ED.  
Accuracy for the lab.

1 Lewandrowski K, Flood J, Finn C, et al. Implementation of point-of-care rapid urine testing for drugs of abuse in the emergency department of an academic medical center: impact on testutilization and ED length of stay. *Am J Clin Pathol.* 2008;129(5):796-801.  
2 Substance Abuse and Mental Health Services Administration, Drug Abuse Warning Network, 2011: National Estimates of Drug-Related Emergency Department Visits. HHS Publication No. (SMA) 13-4760, DAWN Series D-39. Rockville, MD: Substance Abuse and Mental Health Services Administration, 2013.

© 2015 Quidel. All rights reserved.

Quidel | Rapid Diagnostics | [quidel.com](http://quidel.com) | 800.874.1517 | 858.552.1100



SL2901EU | BR944010DEN00 06/18

## Rapid results = rapid decisions.

Rapidly identifying the impact of drug use on the patient's clinical presentation can help reduce their length of stay in the emergency department.



### The Importance of Drug Detection

Clinical Presentation	Importance of Drug Detection
Acute Liver Failure	Acetaminophen overdose is the most common cause of acute liver failure and is often asymptomatic.
Altered Mental Status/ for Psychosis	Identify drugs as a contributing factor; clear patient psychiatric evaluation and placement.
Chest Pain	Avoid the use of beta-blockers for cocaine associated chest pain.
Coma	Identify drugs as a contributing factor or determine appropriate treatment.
Major Trauma	Mind-altering substances may mask a patient's pain level causing injuries to go undetected.
Possible Overdose	Provide appropriate treatment and antidote if needed.
Respiratory Depression	Symptomatic of drug overdose; identify drugs as a contributing factor.
Seizure	Seizures can be caused by drug overdose, withdrawal or other medical conditions; identify drugs as a contributing factor.


## Having answers now matters.

Point-of-care rapid drug screening may improve testing turn-around times (TAT) and length of stay (LOS) in the ED. A prospective evaluation of implementing point-of-care rapid urine drug screening demonstrated a 75 minute savings in TAT which resulted in an average reduction of 3 hours in LOS.<sup>1</sup>

### Result TAT (Turnaround Time)



### Patient LOS (Length of Stay)

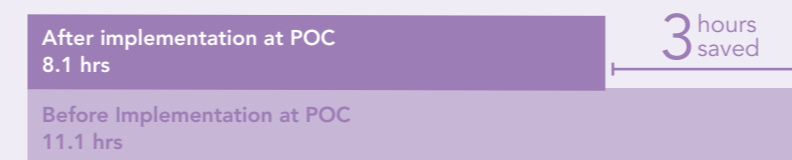


## It's an emergency.

Medical emergencies involving drug misuse or abuse continue to rise, reaching almost 2.5 million in 2011.<sup>2</sup>

Drug screening may be a critical need in your ED.

### Drug-Related Emergency Department (ED) Visits: 2011<sup>2</sup>

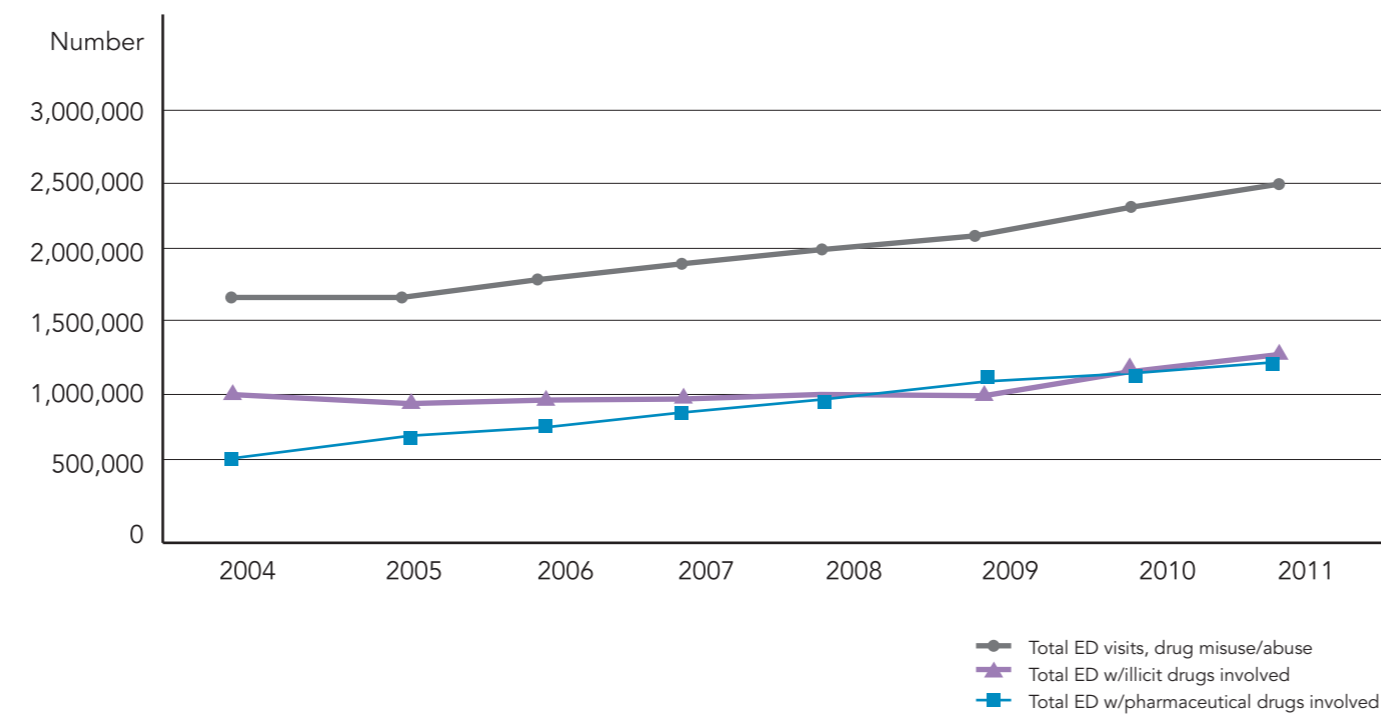

Source: 2011 SAMHSA Drug Abuse Warning Network (DAWN)

## Quidel Triage TOX Drug Screening Products

Cat. #	Description	Kit Size
94400EU	Quidel Triage TOX Drug Screen : 11 Drug Panel APAP•AMP•mAMP•BAR•BZO•COC•MTD•OPI•PCP•THC•TCA	25 Tests
94413	Quidel Triage TOX Drug Screen Control 1 (Negative)	5 x 0.25 mL
94414	Quidel Triage TOX Drug Screen Control 2 (Positive)	5 x 0.25 mL

A matter of life and death.

